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DATEN

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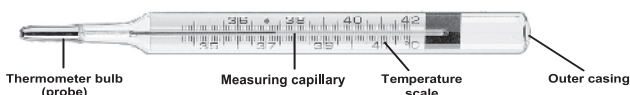
PARTS DESCRIPTION

SPECIFICATIONS

Range: 35,5 °C to 42,0 °C
 Resolution: 0,1 °C
 Accuracy: +0,1 °C / -0,15 °C
 Working Conditions: Temperature: -15 °C to 42 °C
 (It is only possible to measure temperatures that are above the ambient temperature!)

Storage Conditions: Temperature: -15 °C to 42 °C
 Designated Use: Medical liquid-in-glass thermometer for measuring body temperature.
 Description: Analogue maximum thermometer with Galinstan
 Quality Guarantee: The manufacture and testing of the thermometers Geratherm® classic is conducted in compliance with the standards of EU Council Directive 93/42/EEC concerning medical devices
 • of EN 12470-1 - Clinical thermometers, Part 1: Metallic liquid-in-glass thermometers with maximum device.
 The Geratherm Medical AG is certified according to DIN EN ISO 13485 and complies with Council Directive 93/42/EEC. This entitles to affix the CE 0197 marking (Notified body: TÜV Rheinland LGA Products GmbH).

GALINSTAN - A NEW NON-TOXIC MEASURING FLUID - ECO-FRIENDLY
 The mercury-free thermometer, Geratherm® classic, has been developed by Geratherm Medical AG and contains a patented liquid alloy consisting of gallium, indium and tin – called Galinstan. Galinstan is completely non-toxic and environment-friendly.

- PRECAUTIONS**
1. Do not drop the thermometer or subject it to sudden impacts.
 2. Do not bend or bite the tip of the thermometer.
 3. Keep the thermometer out of children's reach.
 4. Thermometers with a damaged outer casing must not be used (risk of injury).
 5. NOTE: Children should be attended by an adult during the entire temperature recording process.
 6. The measuring fluid Galinstan must not be brought into contact with precious or light metals, since it reacts with these metals.
 7. When measuring, please take the necessary care to avoid any danger of injury.
 8. Use of this thermometer is not intended as a substitute for visiting your doctor. Persistent fever, especially in children must be treated by a doctor.

INSTRUCTIONS FOR USE

1. Before taking the patient's temperature, check whether the column of liquid is below 36 °C.
2. Clean the glass thermometer before and after each use. In doing so, please pay attention to the "Notes on Cleaning" below.
3. Position the thermometer on the body according to the method of measurement to be used (axillary, rectal or oral).
4. After about 4 minutes the thermometer will have reached the body temperature. The measuring fluid in the measurement capillary of the thermometer shows the measured body temperature on the temperature scale.
The temperature can be read off by rotating the thermometer gently to and fro.
5. After measuring, the liquid must be returned to the bottom of the scale. To do this, hold the thermometer as close as possible to the upper end and shake it with 15 to 20 rapid wrist movements.

NOTE: Always disinfect the thermometer before and after use. In hospitals, disinfection must be carried out by experienced personnel.

RELIABLE METHODS TO MEASURE TEMPERATURE
ORAL USE - measurement time: min. 4 minutes, normal temperature: approx. 37 °C
 Doctors prefer oral measurement. It provides fast and accurate results. Place the thermometer bulb (probe) in one of the two heat pockets to the left or right side of the tongue.

RECTAL USE - measurement time: min. 4 minutes, normal temperature: approx. 37,6 °C
 Rectal measurement is preferred in the case of children. Insert the thermometer bulb (probe) into the anus (approx. 2 cm). You may use a little skin cream or baby oil on the end of the vessel.
 You may use a probe cover for the thermometer. Insert thermometer into the probe cover. Lubricate the probe cover with a water soluble jelly for easier insertion. Do not use petroleum jelly. Insert tip of probe no more than 2 cm into rectum.

STOP if you meet any resistance.

AXILLARY USE - measurement time: min. 4 minutes, normal temperature: approx. 36,7 °C
 The axillary measurement method provides less accurate measurements than oral and rectal measurement.
 Wipe armpit with a dry towel. Place probe in the patient's armpit and keep the patient's arm pressed firmly against their side.

NOTES ON CLEANING
 The thermometer has a hermetically sealed, hygienic glass casing and can be disinfected without restriction.

The temperature of the cleaning and disinfectant solution must not exceed 42 °C.
 Cold soapy water can be used to clean the glass thermometer before and after use.
 Disinfect the thermometer using a piece of cotton wool or cloth that has been soaked in alcohol, or dip the thermometer in a disinfectant solution (exposure/immersion time as specified by the disinfectant manufacturer).
 If the thermometer breaks, any spilled liquid can be removed using a brush, tissue or cloth dampened in soapy water. The broken pieces may be disposed of as household waste.

WARRANTY
 Subject to normal use, the manufacturer grants an unlimited guarantee for the functionality and accuracy of this thermometer (except in the case of breakage of glass). Damage to the thermometer caused by improper treatment is not covered by the warranty.
Re-calibration is not necessary!

SYMBOL INDEX

	Follow the instructions		Max. and min. temperatures: Store between -15 °C and +42 °C
	Caution		Store in a dry place
	Batch code (mm/yyyy; month/year) on the reverse of the outer casing (applied by means of laser without LOT symbol)		Fragile: handle with care
	Manufacturer		Reference number

Optional: easy flip

Instructions easy flip

The easy flip makes it easier to lead back the measuring liquid of your Geratherm® classic thermometer by considerably less effort when shaking down.

1. Position the thermometer in the easy flip.
2. Make sure that the thermometer is properly secured in the two brackets.
3. Keep the handle of the easy flip between your thumb and index finger.
4. Now, shake the thermometer with short sharp movements back and forth until the liquid is reduced below 36 °C. (See Figure)

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