DE - Gebrauchsanweisung BIOThermare Kalt/Warm Komresse

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EN - Operation Instruction BIOThermare Cold/Warm Compress

This cold/warm compress with seaweed gel filling is reusable and contributes to increasing general well-being through its soothing heat and cold.

SAFETY INSTRUCTIONS: Do not use the cold/warm compress if a person cannot perceive heat and cold properly. Never place the compress directly on the skin, always use the cover provided! Children or people with cognitive impairments should only use the compress under supervision. Use caution to avoid skin damage (e.g. frostbite, chilblains, burns). Remove the compresses immediately if the cold or heat makes you uncomfortable. If gel gets on the skin or in the eyes, rinse the area with plenty of water.

Do not heat the compress in the oven, grill or microwave! If the compress is dirty, wipe it off with a damp cloth. Leaking, damaged or deflated compresses must no longer be used. These can be disposed of with residual waste. With normal use, the compress lasts for at least 1 year. We recommend storing in a polybag in a cool and dry place (max. 25 °C). Protect compress from direct sunlight.

COLD USE: Place the compress in the freezer (-18°C) for at least 2 hours, dry it and place it on the affected area in the provided cover. Only use the compress for as long as you feel comfortable, but no more than 20 minutes. Before applying the compress again, the area to be cooled should remain uncovered for 20 minutes.

WARM USE: Make sure the compress is not damaged before use. Bring water to the boil and remove from heat. Warm the compress in a water bath for approx. 4-5 minutes, carefully remove it, dry it and knead it well. Before applying, carefully test the temperature of the compress. **Caution: The heat may initially be very unevenly distributed. Danger of burns!** The compress usually needs to be reheated after about 30 minutes.

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BIOThermare GmbH